

MEMPHIS® COLLAR

The Memphis® Collar is an innovative cervical collar with high-quality materials that provided an unparalleled combination of immobilization and patient comfort. Removable pads wick away moisture to ensure the patient is clean and comfortable while preventing skin irritation.

Every product we distribute demonstrates our commitment to patient compliance and recovery through superior immobilization and comfort.



Features and Benefits:

- Restricts cervical spine flexion, extension and rotation to promote patient recovery.
 - Latex-free, non-toxic and hypoallergenic laminated foam reduces skin irritation.
 - Adjustable bilateral Velcro® hook and loop fasteners ensure proper fit.
 - Anterior flex tab.
 - Polyethylene Front and Back pieces for improved rigidity and durability.
 - Large trachea opening for quick access to carotid pulse monitoring and emergency tracheotomies.
 - Two-piece design enhances patient comfort.
 - Shoulder and mandible contours increase stability and comfort.
 - Rear panel increases ventilation to reduce heat and moisture while allowing cervical spine palpitation.
 - Easily customized for individual requirements.
 - X-ray, CT and MRI lucent.
 - Clean with mild soap and water.
-

Measurement and Sizing:


Proper sizing of the Collar is important for patient immobilization and comfort. Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

For more information, please refer to Sizes: [Product Table](#)

Proper sizing of the Collar is important for patient immobilization and comfort. Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

Height:

1. Measure the vertical distance from the top of the patient's shoulder to the tip of the chin (Figure 1. "A").
2. The measurement corresponds to the HEIGHT of the Collar. If the patient's measurement falls between two consecutive sizes, apply the smaller size first.

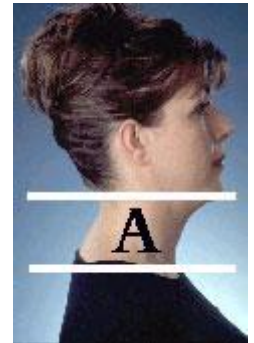

Circumference:

3. Measure around the patient's neck (Figure 2. "B").
4. The measurement corresponds to the size CIRCUMFERENCE of the Collar. If the patient's measurement falls between two consecutive sizes, apply the larger size first.



Application:

Proper application of the Collar is as important as proper sizing for immobilization and patient comfort. The two-piece design consists of a FRONT piece and BACK piece which are packaged as a set. In some cases, an additional replacement pad set is included.

Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

1. After selecting the properly sized Collar, apply the FRONT piece of the Collar to the front of the patient's neck. Center the Collar. The front arrow should point upward.

2. Center the Collar BACK piece on to the patient's neck. With the FRONT piece properly secured, use the bilateral Velcro straps to fasten the FRONT and BACK pieces together.
 3. Improvement of immobilization and comfort is achieved by minor adjustments of the straps and BACK piece supports.
-

Removal:

Before removing the Collar, mark the position of the Velcro strap location to ensure re-application maintains the proper fit.

With the Collar secured on the patient, unfasten the Velcro strap on the RIGHT side and then the LEFT side. Remove the Collar from the patient.

Removable Pad Set:

The Collar has removable laminated pads that wick away moisture to ensure the patient is clean and comfortable while preventing skin irritation. Each set contains three pads; back pad, sternum pad and chin pad.

To properly clean the Collar, it is recommended that the patient has an additional removable pad set as a replacement while the original pad set is air-drying,

Always attach the removable pads in the proper orientation with the pads positioned to contact the patient's skin. Pad adjustment may be necessary to ensure that no plastic contacts the skin.

Cleaning:

Proper hygiene and product cleaning are an important part of a patient's recovery. In addition, it will keep the patient clean, comfortable and prevent skin irritation. A qualified healthcare professional will establish your cleaning routine.

After the Collar is removed, gently clean the patient's neck and face; if there is skin irritation contact a qualified healthcare professional. The patient's neck and face should be dried completely before the Collar is applied.

Separate the Collar's removable pads from the Collar's plastic parts before cleaning. The Collar's plastic parts can be cleaned with mild soap and clean water. The Collar's plastic parts should be dried completely with a cloth or air-dried before it is applied to the patient. While a mild disinfectant is acceptable, do not use harsh chemicals or bleach.

The Collar's removable pads can be cleaned with mild soap and clean water. The Collar's removable pads should be completely air-dried before they are reattached to the Collar's

plastic parts. While a mild disinfectant is acceptable, do not use harsh chemicals or bleach. While the replacement pads are drying, attach an additional pad set to the Collar and apply to patient. If you do not have an additional pad set, please contact a qualified healthcare professional or [contact us](#).

Apply the Collar as instructed by a qualified healthcare professional.



NOTE: This product is a single user device, use only on one patient, do not reuse on multiple patients. Follow a qualified healthcare provider's instructions regarding usage and infection control.

Product Table:

**MEMPHIS® COLLAR
COLLAR**

Item No.	Item Name	Height "A"	Circumference "B"
1116-0000	CHILD - P0	AGE: 0 – 6 MONTHS	
1116-0100	CHILD - P1	AGE: 6 MON – 2 YEARS	
1116-0200	CHILD - P2	AGE: 2 – 6 YEARS	
1116-0300	CHILD – P3	AGE: 6 – 12 YEARS	
1116-1000	SUPER SHORT	1"	10" – 20"
1116-1100	STOUT	1 1/4"	14" – 24"
1116-1200	EXTRA SHORT	1 3/4"	8" – 14"
1116-1300	SHORT	2"	10" – 20"
1116-1400	REGULAR	2 1/2"	10" – 20"
1116-1500	LARGE	3"	10" – 20"

COLLAR & EXTRA PAD SET

Item No.	Item Name	Height "A"	Circumference "B"
1116-0001	CHILD - P0	AGE: 0 – 6 MONTHS	
1116-0101	CHILD - P1	AGE: 6 MON – 2 YEARS	
1116-0201	CHILD - P2	AGE: 2 – 6 YEARS	
1116-0301	CHILD – P3	AGE: 6 – 12 YEARS	
1116-1001	SUPER SHORT	1"	10" – 20"
1116-1101	STOUT	1 1/4"	14" – 24"
1116-1201	EXTRA SHORT	1 3/4"	8" – 14"
1116-1301	SHORT	2"	10" – 20"
1116-1401	REGULAR	2 1/2"	10" – 20"
1116-1501	LARGE	3"	10" – 20"

PAD SET

Item No.	Item Name
1116-0002	CHILD - P0
1116-0102	CHILD - P1
1116-0202	CHILD - P2
1116-0302	CHILD – P3
1116-1902	SUPER SHORT
1116-1102	STOUT
1116-1202	EXTRA SHORT
1116-1902	SHORT
1116-1902	REGULAR
1116-1902	LARGE

COLLAR – FRONT PANEL

Item No.	Item Name	Height “A”	Circumference “B”
1116-0003	CHILD - P0	AGE: 0 – 6 MONTHS	
1116-0103	CHILD - P1	AGE: 6 MON – 2 YEARS	
1116-0203	CHILD - P2	AGE: 2 – 6 YEARS	
1116-0303	CHILD – P3	AGE: 6 – 12 YEARS	
1116-1003	SUPER SHORT	1”	10” – 20”
1116-1103	STOUT	1 1/4”	14” – 24”
1116-1203	EXTRA SHORT	1 3/4”	8” – 14”
1116-1303	SHORT	2”	10” – 20”
1116-1403	REGULAR	2 1/2”	10” – 20”
1116-1503	LARGE	3”	10” – 20”

COLLAR – BACK PANEL

Item No.	Item Name	Height “A”	Circumference “B”
1116-0004	CHILD - P0	AGE: 0 – 6 MONTHS	
1116-0104	CHILD - P1	AGE: 6 MON – 2 YEARS	
1116-0204	CHILD - P2	AGE: 2 – 6 YEARS	
1116-0304	CHILD – P3	AGE: 6 – 12 YEARS	
1116-1004	SUPER SHORT	1”	10” – 20”
1116-1104	STOUT	1 1/4”	14” – 24”
1116-1204	EXTRA SHORT	1 3/4”	8” – 14”
1116-1304	SHORT	2”	10” – 20”
1116-1404	REGULAR	2 1/2”	10” – 20”
1116-1504	LARGE	3”	10” – 20”